
CERAGEM


Head Office | 10, Jeongja 1-gil, Seonggeo-eup, Seobuk-gu,
/Factory | Cheonan-si, Chungcheongnam-do, 31045, Korea
TEL : (82)41-523-2550
FAX : (82)41-523-2560
Homepage | www.ceragem.net

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Gebrauchsanweisung | CGM EB-1901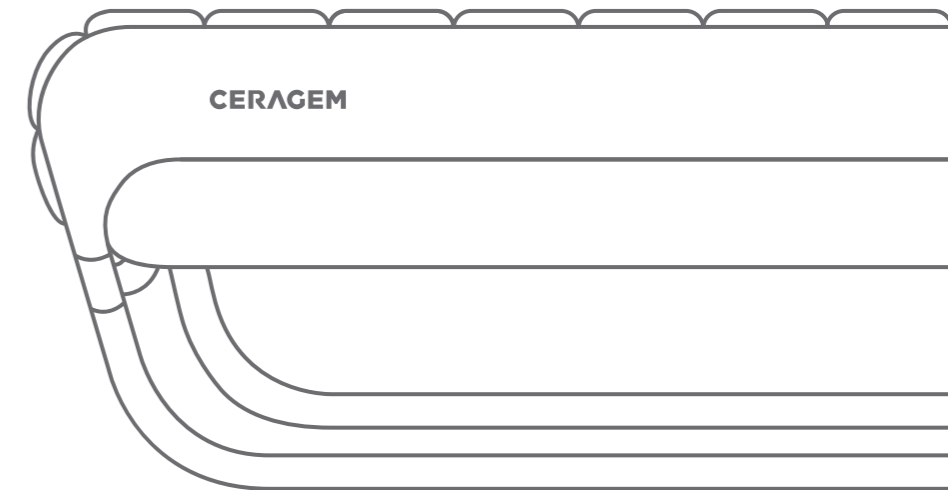

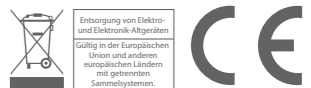V4KN073430

Important safety issues

This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.

The information below stands to protect the safety of the user and to prevent any property damage. All users must read the manual and use the product appropriately. The important safety issues are divided into "Danger", "Warning" and "Caution" and their meanings are as follows. This picture is a symbol used to warn in case of dangerous situations or dangerous control methods.

 Danger	In case of violating the instructions, there is an emergent possibility of serious damage or even death.
 Warning	In case of violating the instructions, there is some possibility of serious injury or death.
 Caution	In case of violating the instructions, there is possibility of minor injury or property damage.

Please follow the instructions carefully in order to reduce potential electric shocks.

1. While using this product, do not use pins or other metals.
2. Please check the frame before use. If there are any signs of cracks, breaks or expiration, stop using the product.
3. Do not use the product when there is humidity or water.
4. The external projector must be used with the external projector cover so that there is no direct contact between the product and skin. (There is risk of receiving a burn.)
5. To prevent damage to the eye (eyeball), do not place the external projector directly on the eye.
6. In case of thunder, lightning or an earthquake, please stop using the product and unplug it. (There is risk of receiving an electric shock or a burn.)
7. Do not pull the cord to unplug or touch the power plug with wet hands. (There is risk of receiving an electric shock or a burn.)
8. Apart from the mode for massaging the abdomen or the manual mode, do not use the product laying face down. (If the product is used lying down, do not use near the rib cage and only on the abdomen.)

Important safety issues

Warning

1. Only use the product if it is in mechanically complete condition.
(If damaged products are used, there is a possibility of injury or burns.)
2. If you have finished using the product or will not use it for a long time, unplug the product.
3. Apart from the external projector cover provided by our company, do not place other products or substances on top that may cause fires (such as fiber products, blankets and etc.)
(Overheating can cause fires and electric shocks.)
4. In case of use by a disabled person or children, guardians must be present.
*Warning! Keep children away from where this product has been installed.
*Warning! Make sure that children do not jump on the thermal massage bed.
5. Do not use the product if it is in the conditions mentioned below. Call the customer center and request inspection or after-sales service.
 - If the product has been dropped or is damaged
 - If the product is wet
 - If the cord or the power button is damaged. (There is a risk of an electric shock or a fire.)
6. Keep the power cord away from electric heating appliances or heaters.
7. This product has only been designed for indoor use. Do not use it outdoors.
8. Do not use the product where there are foreign substances in the air or places that handle oxygen.
9. Do not use or apply skin care, cosmetic or medical products on your skin before and during the use of this product.
10. When dismantling the product, make sure both the remote control and the main system are switched "Off" before unplugging.
11. Be cautious of heated surfaces. (Can cause severe burns.)
12. Do not use the external projector on parts of the body that have low levels of sensitivity or parts that have reduced blood flow.
(The heat remaining from the product may be dangerous to children or those who are in bad health.)
13. Please do not sleep on the product.
14. Do not randomly disassemble, repair, or remodel the product. Call the customer center when in need of repair.
(There is a risk of electric shock, fire, injury, and burns.)
15. Please take care not to insert your hand or any other body part into the main product or supporting mat. (There is a risk of electric shock and injury.)
16. To avoid the risk of injury:
 - Pregnant women, people who have malignancies, high fever, osteoporosis, people in poor physical health, people in poor physical shape or undergoing any medical treatment, and people suffering back, neck, shoulder or hip pain should consult their doctor before using the product.
 - If you have a pacemaker, defibrillator, other artificial implants or other medical devices, consult with your doctor prior to use.
 - If you have sensitive reactions or allergy to heat, you should also consult your doctor before using the product.
 - Do not use on bare skin; always wear clothing.
 - Do not use if you have had any major surgeries, including back and/or hip surgeries.
 - Consult your doctor prior to use if you have any external wound(s) or infection(s) on the skin where the product will come into direct contact with your skin.
 - Immediately stop using the product if you start to feel sick or uncomfortable, or if the massage seems too painful.

Important safety issues

- As of now, there is no evidence to suggest that this product interferes with or influences medications. Those taking any prescribed medications should consult their doctor before using this product.
 - Do not go to sleep while using the product.
 - Do not use the product after drinking alcohol.
 - Do not allow children or pets to play on or around the product, especially during operation.
 - Circulatory insufficiency or weak circulation in legs or heart diseases.
 - Pain without cause defined.
17. Using the cable incorrectly may lead to strangulation. Please be careful.
There has been no evidence of this product interfering with or influencing medications. If taking any medications, consult your doctor before using this product.

Caution

1. When using the internal projector, do not twist your body and use it with a straight posture.
2. When the product is set to high temperatures, there is a risk of burns, so set the projector's temperature at an optimal level. (Depending on the physical constitution or thickness of skin, use the product at optimal temperature.)
3. Those with sensitive skin must use the product at lower temperatures. (There is a risk of burns)
4. Children, those who cannot control the temperature by themselves, and those who have weak sensory skin must be cautious of using the product for an extended period even if it is set at a low temperature because there is risk of receiving a burn.
(Warning! Please use while wearing clothing.)
5. If there are any abnormal symptoms while using the product, you must consult a doctor before using it again.
6. Do not bend the power cord or the outer wiring by force.
(There is a risk of electric shocks and fires.)
7. Do not place flammable objects (candles, cigarettes and alcohol), objects with water in them, heavy objects or sharp objects on top of the product.
(They can cause electric shocks and damage.)
8. When climbing on or off the product, be cautious of the risk of falling.
9. Use according to the instruction manual and keep to the duration and methods instructed in the manual.
(Use 1~2 times a day and do not use other products unless supplied by the manufacturer.)
10. Do not use the external projector(s) for more than 2~3 minutes on one part of the body.
(The projector must be moved to another part of the body after 2~3 minutes.)
11. Do not apply shock to the product.
12. Do not close the supporting mat or detach the remote control during use.
(The active mode is automatically terminated.)
13. The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance.
14. Please take care not to spill any foreign substance in between the main product and the supporting mat.
(It can cause electric shock, fire, injury, and product failure.)
15. Please do not throw the remote control or pull on the connecting cable.
(It can cause injury and product failure.)
16. The product must not be located in a location where it is hard to remove the detachable power plug.

Important safety issues

17. When a set period of time has passed after setting the mode, the internal projector will move automatically. Lie down in the correct posture.
18. If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a hazard.

Notice to users

This product complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this product may not cause harmful interference, and (2) this product must accept any interference received, including interference that may cause undesired operation.

NOTE: This product has been tested and found to comply with the limit for a Class B digital product, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation, which can be determined by turning the product off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the product and receiver.
- Connect the product into an outlet on a circuit different from that to which the receiver is connected.

Caution

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the product.

Contents

>> Important safety issues	01
- Danger	
- Warning	
- Caution	
I. Introduction of the product	07
1. Product's characteristics	
2. The name of each part	
II. Installation and preparation	11
1. Check product's Composition	
2. Check before product installation	
3. Installation	
4. Check after installation	
III. Using the product	16
1. Check before using the product	
2. Remote control layout	
3. How to use the remote control	
4. How to move the product	
IV. Management and matters that require attention	37
1. Self inspection	
2. Management techniques and storage methods	
3. Check before A/S request	
4. Diagnosis code message	
5. Product specifications	
6. Product warranty	

MEMO

Dotted lines for writing.

I . Introduction of the product

1. Product's characteristics
2. The name of each part


1. Product's characteristics


2. The name of each part

Main body

* Design (color) and the cover pattern may vary according to the product.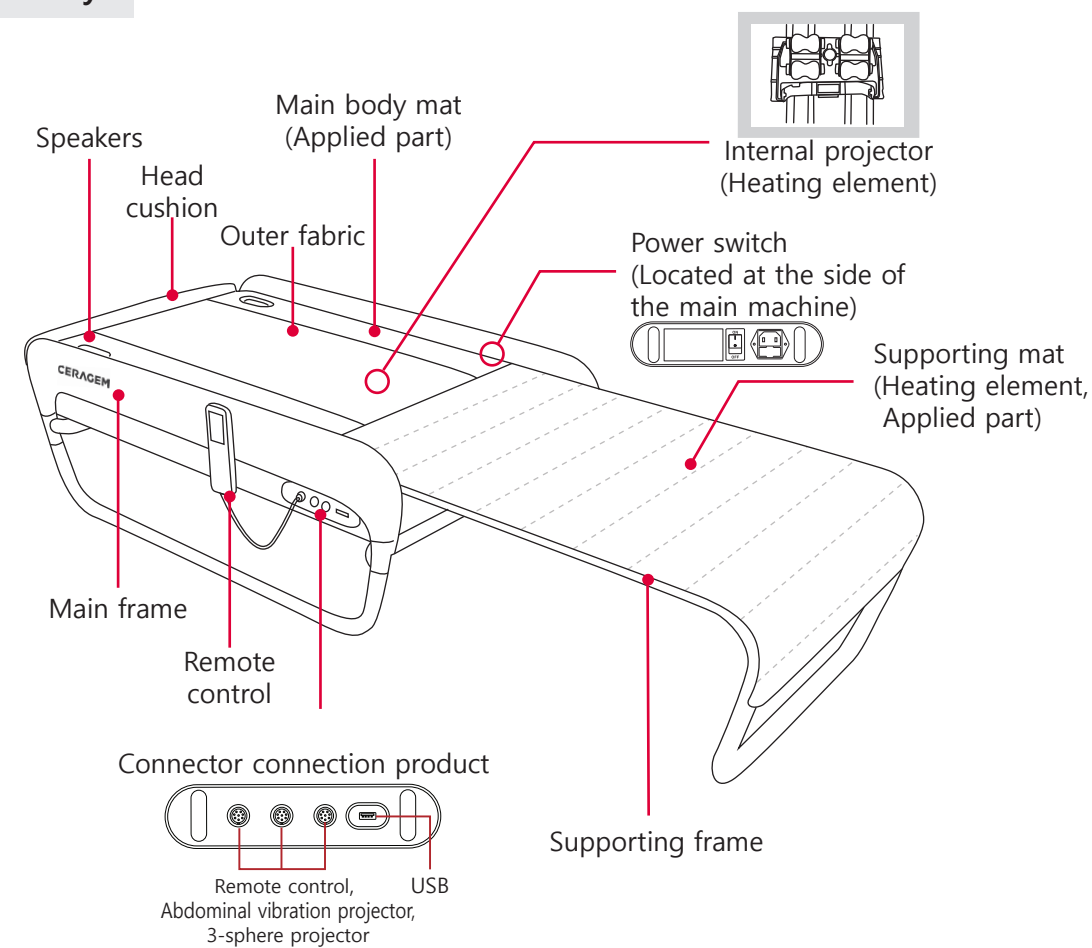

Connecting parts

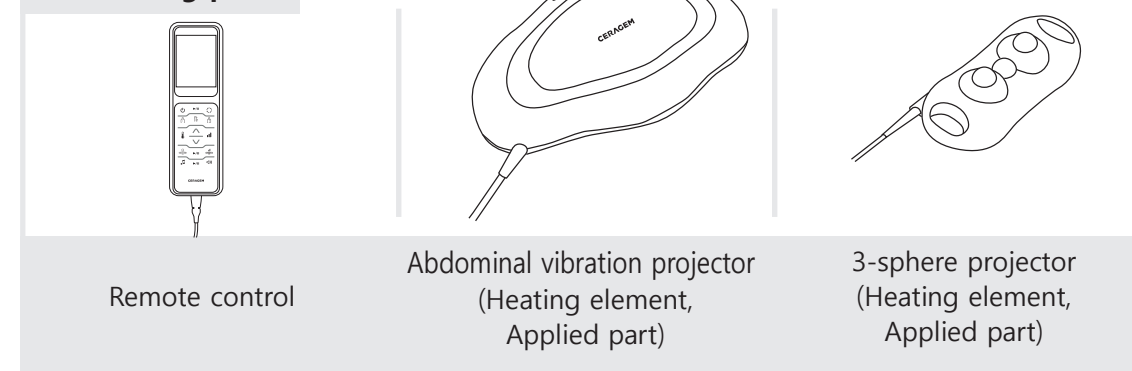What is a mounting product?

A part which comes into contact with the user's body when using the product.

1. Check product's composition

1	3-sphere projector Easy to use on body parts such as the abdomen, shoulders, arms, legs and etc. Can be used with the main product or shared between two people.
2	Abdominal vibration projector This can be connected to the main body and used on the abdomen.
3	Projector covers (abdominal vibration projector and 3-sphere projector) Can be used to cover the external projector.
4	Remote control Used to check the product functions and operating condition of the product.
5	Remote control back label Explanation of the buttons on the remote control (attaches to the back of the remote control).
6	Remote control cradle(Bolt) The remote control cradle can be assembled onto and used on the main body of the product.
7	Power cord Connects with the main body and plugs into an outlet for use.
8	User manual Explains use of the product.
9	Outer fabric Protects against product contamination and burns; this detachable fabric makes washing easy.
10	Head cushion This is attached to the top of the product.
11	Assembly guide Product assembly methods are described.

2. Check before product installation

- 1 Install where the ground is flat to avoid the risk of damaging the product.
- 2 Try and avoid installation where there is a lot of humidity or dust.
(To avoid electric shocks or fire.)
- 3 Install according to the designated order.
- 4 When installing, check that the rated voltage is 220-240V~.
(To avoid circuit damage or fire.)
- 5 Do not drop the product or apply any force to it.
(This could cause malfunctioning.)
- 6 Do not use damaged power cords, plugs or old sockets.
- 7 After you purchase the product, a professional installation technician will install it. Do not assemble or disassemble the product yourself.


3. Installation

* After you purchase the product, a professional installation technician will carry out the basic assembly process.


- 1 Take out the product and its components from the box.
- 2 After fastening the supporting mat to the sliding rail, place the main body mat (which is attached to the head cushion and the outer fabric) onto the main body frame. (After moving the supporting frame to the middle of the main body, push in the main body mat completely.)
- 3 Connect the cable which is attached to the main body to the connector of the supporting frame. (Open the supporting frame before connecting.)
- 4 Assemble the remote control cradle to the frame and attach the remote control to the cradle.
- 5 Check the remote control, abdominal vibration projector and 3-sphere projector connector, and connect them correctly.
- 6 Before connecting the power cord, connect the remote control first.
- 7 Connect the power cord to the main body and plug into a socket.

4. Check after installation

- 1 Turn on the power at the side of the main body. (Please turn off the power when the product is not being used.)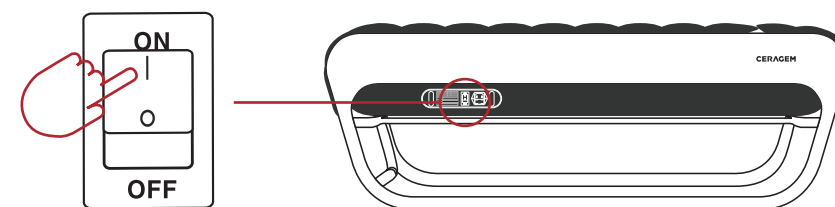

- 2 Press the power key of the remote control and check whether the display turns on.

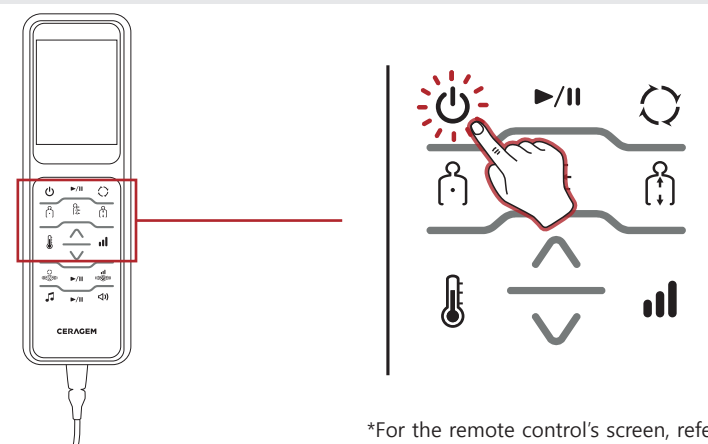*For the remote control's screen, refer to page 20.

Caution

1. According to the user, set the temperature appropriately when using the product.
2. When sitting down and using the product, open the supporting frame completely.
3. After using the product, turn off the power on the remote control and then turn off the main body.

III. Using the product


1. Check before using the product
2. Remote control Layout
3. How to use the remote control
4. How to move the product

1. Check before using the product


Wait!


Before using the product, check if the remote control and external projectors are connected correctly.
(An image is displayed on the remote control only when external projectors are connected.)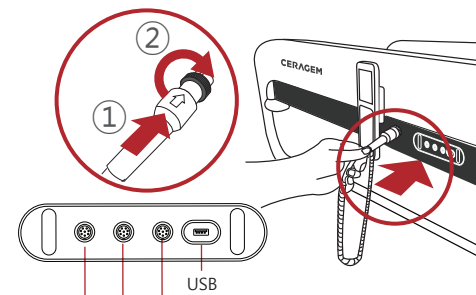

- 1 Connect the remote control to the main body.
(Insert the connector correctly and fix it by turning the fixture.)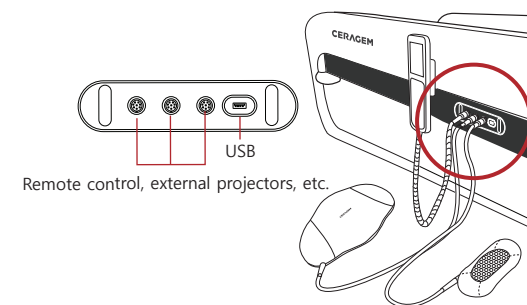


- 2 Connect the external projectors (abdominal vibration projector, 3-sphere projector) to the main body in the same way as with the remote control.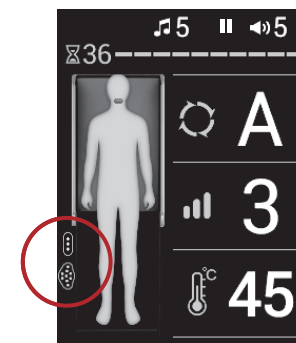

- 3 After turning on the main body, check if external projector (abdominal vibration projector, 3-sphere projector) icons are displayed on the screen of the remote control.


2. Remote control Layout

Buttons on the remote control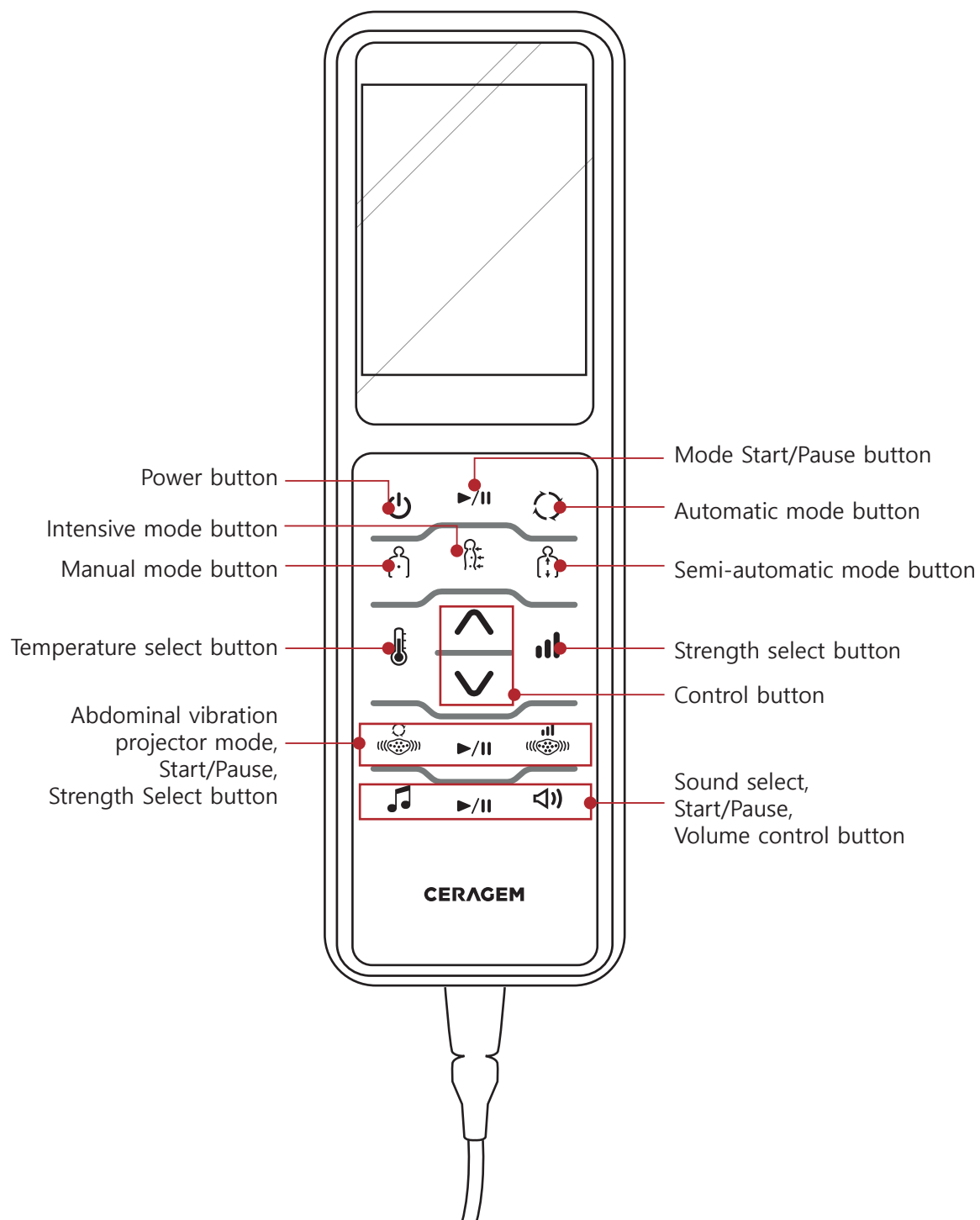


2. Remote control Layout

Screen display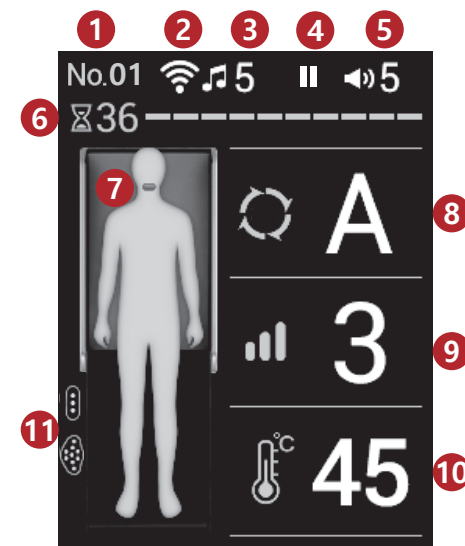


- Main screen -


1	product no.
2	Internet connection (when connected icon)
3	Sound source (Ceragem sound / USB / Bluetooth / Voice guidance)
4	Sound pause
5	Volume(0~20)
6	Time remaining (in minutes)
7	Projector location
8	Mode (Automatic mode A,1~9/Intensive/Semi-automatic/Manual)
9	Strength display (1~9)
10	Internal projector temperature (30~65°C)
11	Display of connection of the 3-sphere projector and the abdominal vibration projector

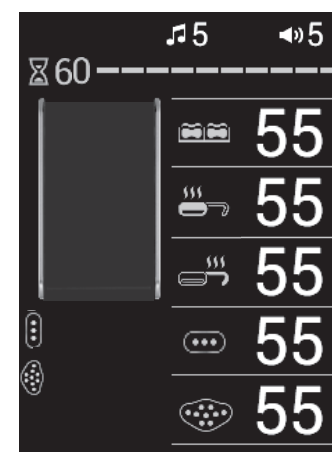When using the thermal sofa

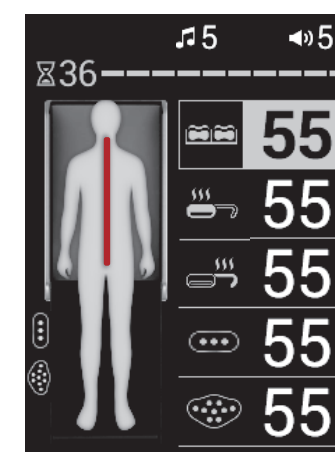When setting the temperature

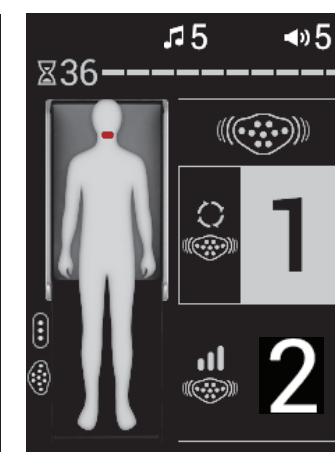
When setting the vibration projector

Wait!

- When the supporting mat is closed, an image of the closed supporting mat will be displayed.
- When 10 seconds have passed after the remote control was last operated, the screen will become dark. (When a button is operated, the screen will become bright again.)

3. How to use the remote control


Turning on the product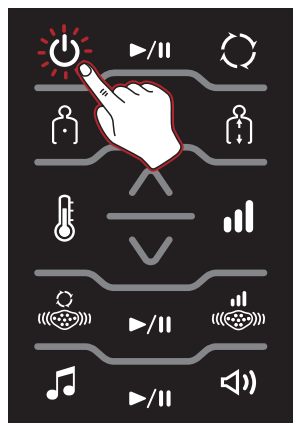

1. Check the power connection to the main body, and then press the power button on the remote control.

- To turn off the product, press the Power button on the remote control for approximately 2 seconds.

Wait!

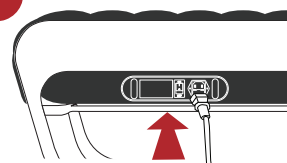
Before using the product, check if the power cord is connected to the outlet correctly.

If there is any discomfort or emergency while operating the product, press the on/off button on the remote control to stop the operation.

2. The Ceragem logo will appear on the screen of the remote control and the remote control will turn on.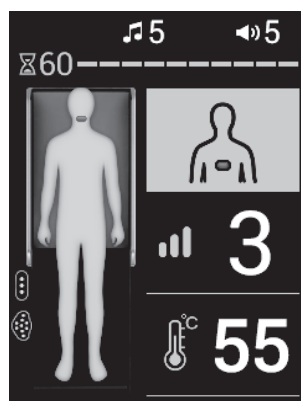

3. When the main screen is displayed, use the product by pressing the desired button.


- The last-used set value will be displayed.
(The set value will not be stored if the product is turned off abnormally.)

Wait!

- When the product is closed, the closed product image will be displayed.

3. How to use the remote control

Setting the temperature

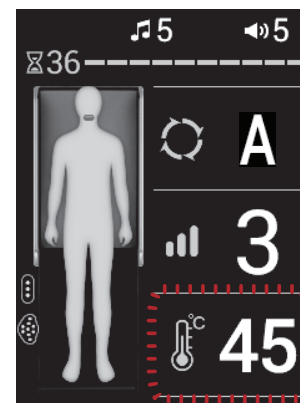1. Check the current temperature and then press the temperature select button.

- The current temperature of the internal projector will be displayed on the main screen.

Wait!


- When the product is closed, the closed product image will be displayed.

2. Select a desired item by pressing the temperature select button.

- Temperature selection order: Internal projector → Main body mat → Supporting mat → 3-sphere projector → Abdominal vibration projector
- When a temperature is selected, the image of the relevant item will be displayed in red.
- The abdominal vibration projector and 3-sphere projector will only be displayed when connected and the position may vary according to the installation order.

Internal projector	Main body mat	Supporting mat	3-sphere projector	Abdominal vibration projector
Off, 30~65°C	Off, 30~60°C			

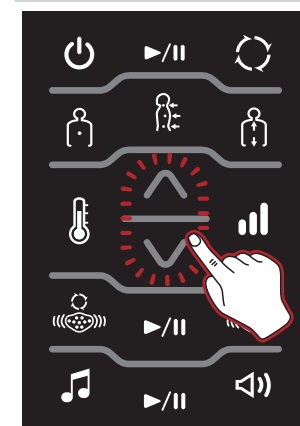3. Use the product after selecting a temperature and adjusting it by using the up/down button.

- Shortly after the setting is complete, the screen will return to the main screen.


Caution

Do not use the external projectors (abdominal vibration projector, 3-sphere projector) on any part of the body for an extended period of time.
(There is risk of receiving a burn, even at a low temperature.)

3. How to use the remote control

Setting the strength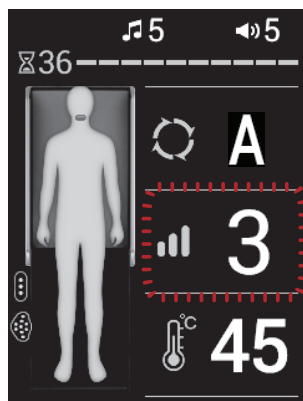


1. Check the current strength and press the strength button.

- The current strength of the internal projector will be displayed on the main screen.


Wait!

- When the product is closed, the closed product image will be displayed.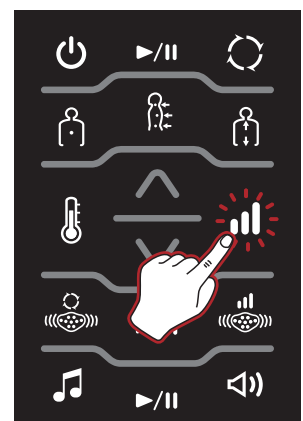

2. Set the strength as desired by pressing the strength button or the up/down button while the relevant item is displayed in yellow.


- Set the desired strength by using the strength button or the up/down button while the relevant item is displayed in yellow.
- Strength can be set at Levels 1 ~ 9.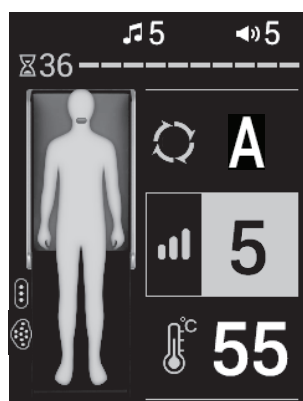

3. Check the changed strength setting on the screen of the remote control.

3. How to use the remote control

Setting the thermal sofa mode (supporting mat closed)

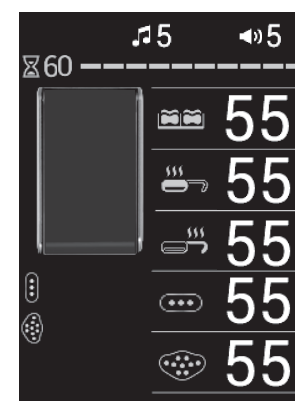1. Turn on the remote control and check the screen.


- The abdominal vibration projector and 3-sphere projector will only be displayed when connected and the position may vary according to the installation order.

2. Select a desired item by pressing the temperature select button, and set the desired temperature by pressing the up/down button.

- When a temperature is selected, the image of the relevant item will be displayed in red.
- When the thermal sofa mode is used, the temperature of the internal projector and the main body mat (preheating) can be adjusted.

Internal projector	Main body mat	Supporting mat	3-sphere projector	Abdominal vibration projector
60	55	55	55	55
55	60	55	55	55
55	55	60	55	55
55	55	55	60	55
55	55	55	55	60
Off, 30~65°C	Off, 30~60°C			

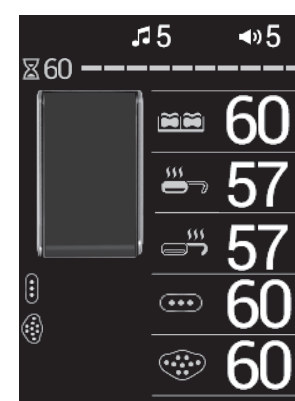3. After setting the temperature, the current temperature will be displayed.

- If the remote control is operated while using the product, the time remaining will be initialized.


Caution


Do not use the external projectors (abdominal vibration projector, 3-sphere projector) on any part of the body for an extended period of time.
(There is risk of receiving a burn, even at a low temperature.)

3. How to use the remote control

Using the sound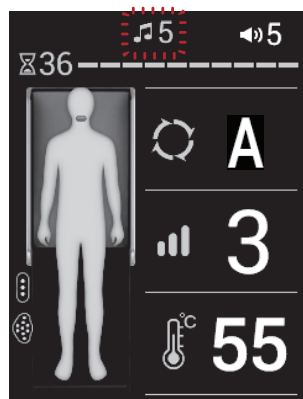



1. Check the current sound status.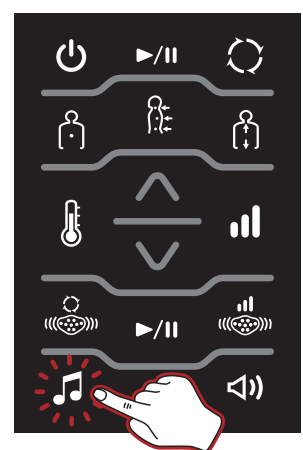

2. Press the Sound select button repeatedly to select the desired sound, and set the desired value by using either the play or the up/down button.

- If a USB is connected but there is no sound source, the icon will not be displayed.
- If Bluetooth and USB is connected, the corresponding icon will be displayed.
- When the sound is paused, or 0 is set for the volume while voice guidance is being set, voice guidance will be deactivated.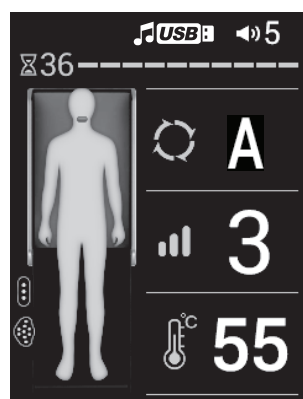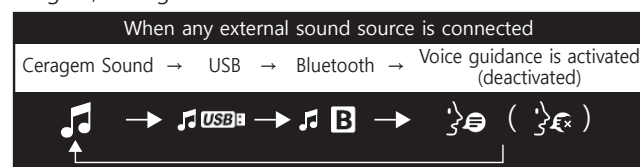

3. Check the changed sound status on the screen of the remote control.


- File recognition time may be delayed according to the USB specification or storage capacity.

Wait!


- When the Main mode is paused, the sound will be paused also. The sound will play back when the Sound Start button is pressed again.
- When the sound is paused while using the product, the pause image will be displayed.
- Noise may be heard as a result of electromagnetic waves.

3. How to use the remote control

Using the volume

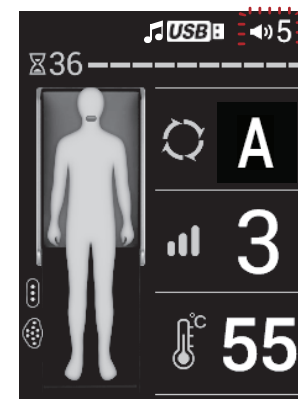
1. Check the current volume status.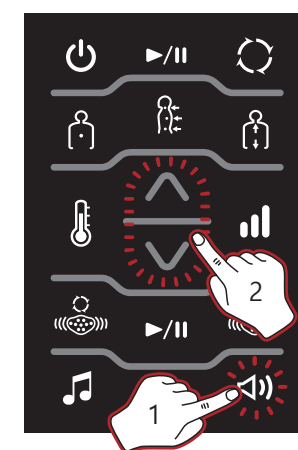

2. Set the volume as desired by pressing the Volume button repeatedly or the up/down button.

- When setting the sound status, the figures for the relevant item will be displayed in yellow; the desired sound and volume may be set at this time.
- Volume can be set at Levels 0~20.
- The volume of voice guidance can be adjusted using the Volume button only when the setting is selected.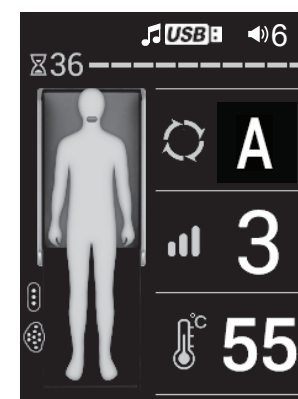


3. Check the changed volume status on the screen of the remote control.

Wait!

- Noise may be heard as a result of electromagnetic waves.

3. How to use the remote control

Using abdominal vibration projector mode

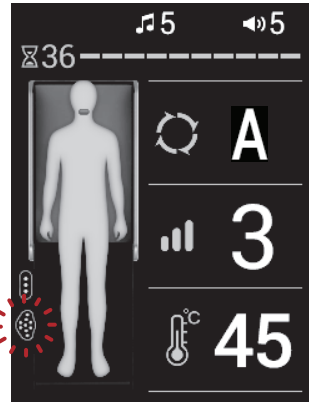1. Check if the abdominal vibration projector is connected correctly.
 - The current status will be displayed
 - The external projector icon will only be displayed when connected.


Wait!

- When the product is closed, the closed product image will be displayed.

2. Set the mode by pressing the abdominal vibration projector mode button and press the start/pause button.
 - Set the desired mode by using the Vibration projector mode button or the up/down button while the relevant item is displayed in yellow.
 - Modes 1 ~ 3 can be used.

	Sequential press (Modes 1~3 are selected sequentially)
Mode 1~3	Approximately 18 minutes
	- Do not operate the abdominal vibration projector on the floor. (The surface of the product may be damaged.)

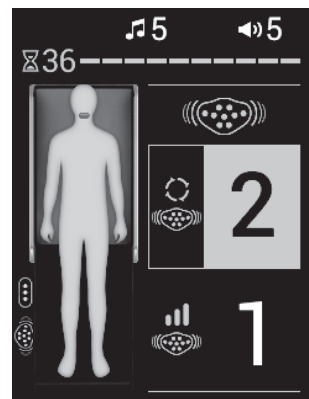
3. After setting the mode, operation will start in the selected mode.
 - Shortly after the setting is complete, the screen will return to the main screen.

Wait!


- When the Main mode is paused, the abdominal vibration mode will be paused also, and will resume when the abdominal vibration Projector Start button is pressed.
- When the abdominal vibration projector mode is paused while using the product, the pause image will be displayed.

3. How to use the remote control

Using abdominal vibration projector strength

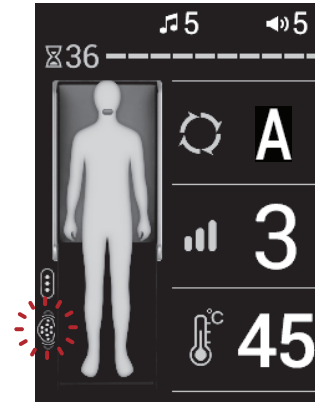
1. Check the screen of the remote control.
 - The current status will be displayed.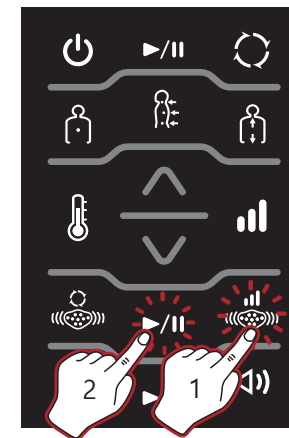

2. Set the strength by pressing the abdominal vibration projector strength button and press the start/pause button.
 - You can set the desired mode by using the Vibration projector strength button or the up/down button while the relevant item is displayed in yellow.
 - Strength can be set at Levels 1~2.

	Sequential press (Strength 1~2 are selected)
--	--

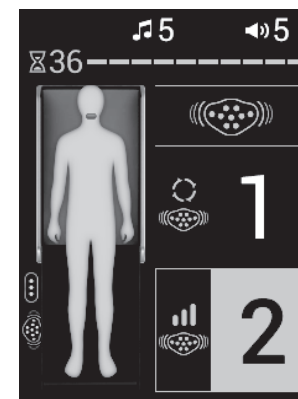
3. After the setting ends, the mode will start at the selected strength.
 - Shortly after the setting is complete, the screen will return to the main screen.

Caution

Do not use the abdominal vibration projector on any part of the body for an extended period of time.
(There is risk of receiving a burn, even at a low temperature.)


3. How to use the remote control

Using automatic mode

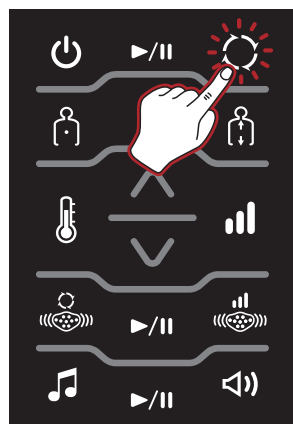1. Turn on the remote control, check the current status and press the automatic mode button.


Mode A : Approximately 36 minutes and 30 seconds.
 Mode 1 : Approximately 75 minutes.
 Mode 2 : Approximately 18 minutes.
 Mode 3 : Approximately 36 minutes and 30 seconds.
 Mode 4 : Approximately 36 minutes and 30 seconds.
 Mode 5 : Approximately 36 minutes and 30 seconds.
 Mode 6 : Approximately 36 minutes and 30 seconds.
 Mode 7 : Approximately 36 minutes and 30 seconds.
 Mode 8 : Approximately 36 minutes and 30 seconds.
 Mode 9 : Approximately 36 minutes and 30 seconds.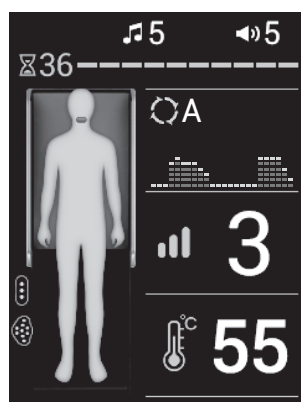

2. Select a mode by pressing the automatic mode button. body shape sensing will begin. Lie down in the correct posture.

- You can set the desired mode by using the automatic mode button or the up/down button while the relevant item is displayed in yellow.
- When the mode begins, the Ceragem sound will be played.
- The mode cannot be paused during body type sensing.
- The body type graph varies from person to person.

 Sequential press (Mode A, 1~9 are selected sequentially)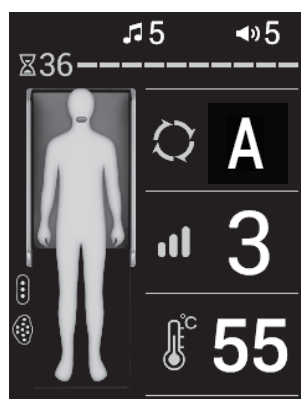

3. When body shape sensing ends, the mode will begin automatically. Maintain correct posture.

- The time remaining will be displayed at the top of the human body icon.


Wait!

- When pausing, the internal projector will move down vertically in order to prevent burns and injury.
(When the mode is executed again, the set strength will be restored.)
- When 5 minutes have passed after pausing, the product will turn off automatically.


3. How to use the remote control

Using intensive mode

Duration for mode Approximately 90 minutes.

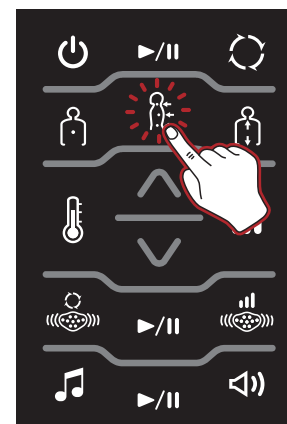
1. Turn on the remote control, check the current status and press the intensive mode button.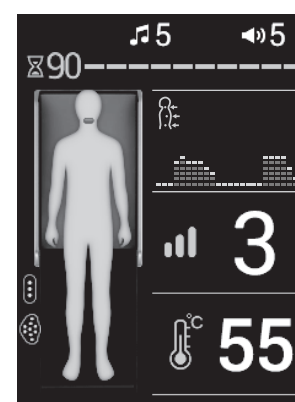

2. When body shape sensing ends, the intensive mode will begin. Maintain correct posture.

- The time remaining will be displayed at the top of the human body icon.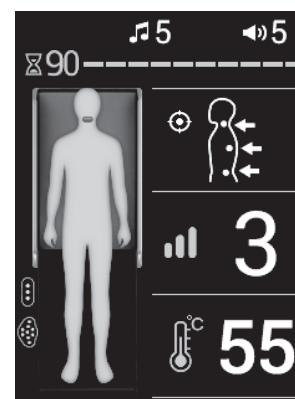

3. If you wish to set a specific position for using the product, use the product after moving the internal projector to a desired position by using the up/down button.

- If you cancel the position setting by pressing the intensive mode button again, the internal projector will move.
- When 90 minutes have passed after setting the remote control, the remote control will turn off automatically and the mode will end.

Wait!

- When pausing, the internal projector will move down vertically in order to prevent burns and injury.
(When the mode is executed again, the set strength will be restored.)
- When 5 minutes have passed after pausing, the product will turn off automatically.


3. How to use the remote control

Using manual mode


Duration for mode Approximately 60 minutes.

1. Turn on the remote control, check the current status and press the manual mode button.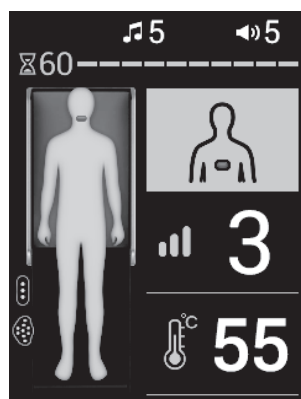

2. Move the internal projector to the desired position using the up/down button.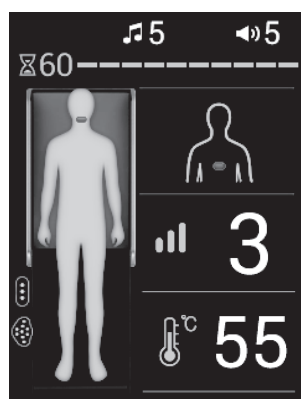

3. The manual mode will begin at the designated position. Maintain correct posture.
 - The time remaining will be displayed at the top of the human body icon.
 - If the remote control is operated while using the product, the time remaining will be initialized.
 - When 5 minutes have passed after setting the position, the internal projector will move automatically in order to prevent burns.
 - When 60 minutes have passed after operating the remote control, the remote control will turn off automatically and the mode will end.


Wait!

- When pausing, the internal projector will move down vertically in order to prevent burns and injury. (When the mode is executed again, the set strength will be restored.)
- When 5 minutes have passed after pausing, the product will turn off automatically.


3. How to use the remote control

Using manual mode (master mode)

Duration for mode Approximately 60 minutes.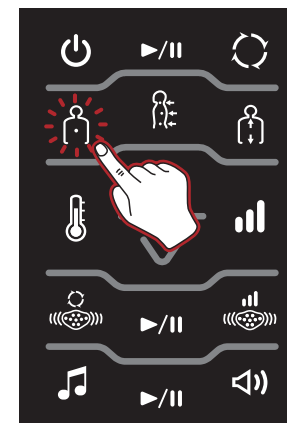


1. Turn on the remote control, check the current status and press the manual mode button twice.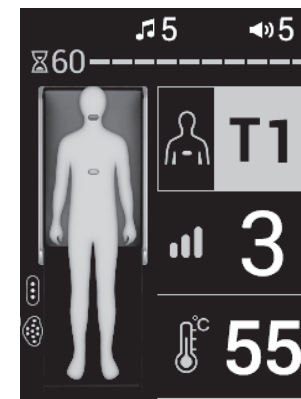

2. Move the internal projector to the desired position using the up/down button.

- The selected position will be displayed in green.

Position	Screen display (24 segments)
Neck area(cervical spine)	C1~7
Back area(thoracic vertebrae)	T1~12
Waist area(lumbar vertebrae)	L1~5

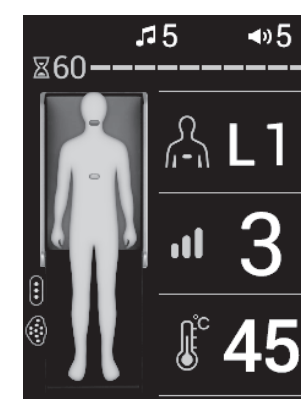3. When body shape sensing ends, the manual master mode will begin. Maintain correct posture.

- The time remaining will be displayed at the top of the human body icon.
- If the remote control is operated while using the product, the time remaining will be initialized.
- When 5 minutes have passed after setting the position, the internal projector will move automatically in order to prevent burns.
- When 60 minutes have passed after operating the remote control, the remote control will turn off automatically and the mode will end.

Wait!


- When pausing, the internal projector will move down vertically in order to prevent burns and injury. (When the mode is executed again, the set strength will be restored.)
- When 5 minutes have passed after pausing, the product will turn off automatically.

* In semi-automatic mode, the internal projector will not move to the cervical spine after the mode ends.


3. How to use the remote control

Using semi-automatic mode


Duration for mode Approximately 60 minutes.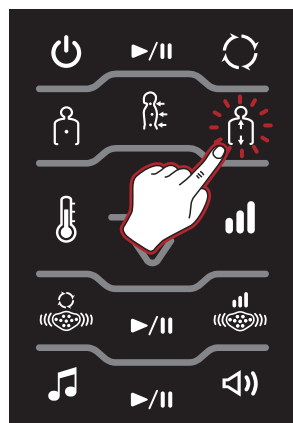

1. Turn on the remote control, check the current status and press the semi-automatic mode button.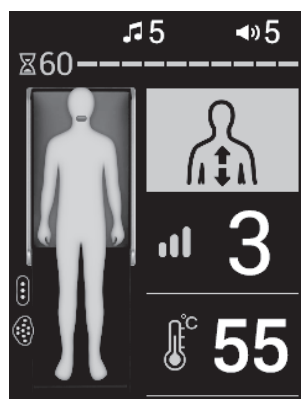

2. Move the internal projector to the desired position using the up/down button.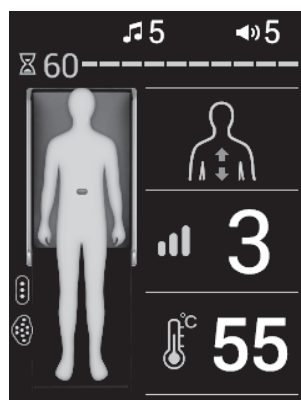

3. The semi-automatic mode will begin at the designated position. Maintain correct posture.

- The time remaining will be displayed at the top of the human body icon.
- If the remote control is operated while using the product, the time remaining will be initialized.
- When 60 minutes have passed after operating the remote control, the remote control will turn off automatically and the mode will end.

Wait!


- When pausing, the internal projector will move down vertically in order to prevent burns and injury.
(When the mode is executed again, the set strength will be restored.)
- When 5 minutes have passed after pausing, the product will turn off automatically.

* In semi-automatic master mode, the internal projector will not move to the cervical spine after the mode ends.


3. How to use the remote control

Using semi-automatic mode (master mode)

Duration for mode Approximately 60 minutes.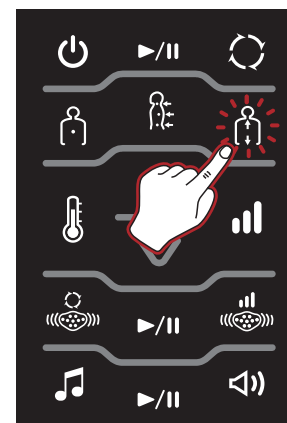


1. Turn on the remote control, check the current status and press the semi-automatic mode button twice.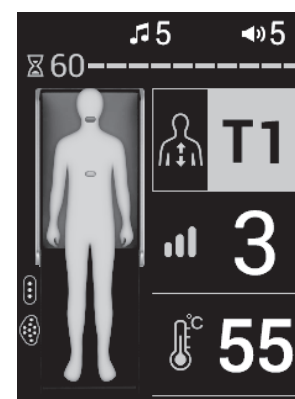

2. Move the internal projector to the desired position using the up/down button.

- The selected position will be displayed in green.

Position	Screen display (24 segments)
Neck area(cervical spine)	C1~7
Back area(thoracic vertebrae)	T1~12
Waist area(lumbar vertebrae)	L1~5

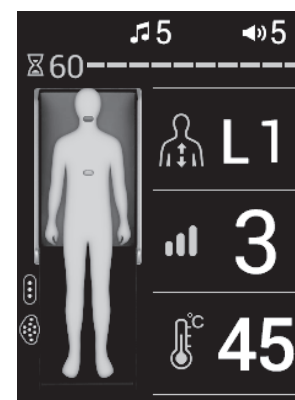3. When body shape sensing ends, the Semi-automatic master mode will begin. Maintain correct posture.

- The time remaining will be displayed at the top of the human body icon.
- If the remote control is operated while using the product, the time remaining will be initialized.
- When 60 minutes have passed after operating the remote control, the remote control will turn off automatically and the mode will end.


Wait!

- When pausing, the internal projector will move down vertically in order to prevent burns and injury.
(When the mode is executed again, the set strength will be restored.)
- When 5 minutes have passed after pausing, the product will turn off automatically.


3. How to use the remote control

Changing a product number


- * If there are two or more products in one space, the product number can be changed as required for use of the product.
- * When Bluetooth or the Application is connected, Search using the set product number is available.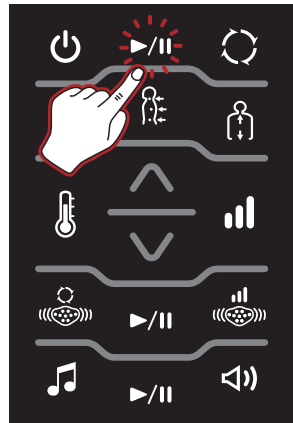

1. Close the supporting mat and press and hold the start/pause button for about 3 seconds.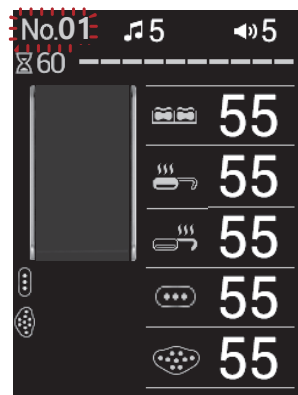

2. The current product number appears on the upper left side of the screen.

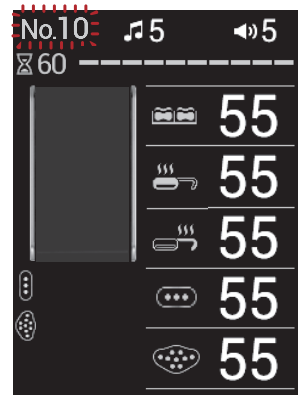
3. Change the product number to the desired number using the up/down button.

- The product number will only appear when setting.
- The product number can be set between No. 01~99.


3. How to use the remote control

Connecting to the internet


- * Download and use an application provided by ceragem from an application market.
- * The IoT App. may not be supported in all countries.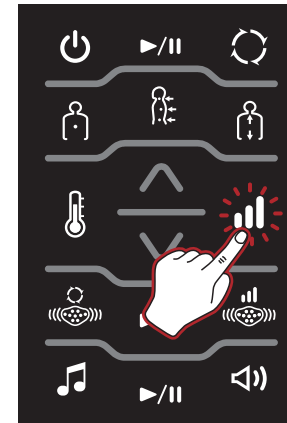

1. Close the supporting mat and press and hold the Strength button for about 3 seconds.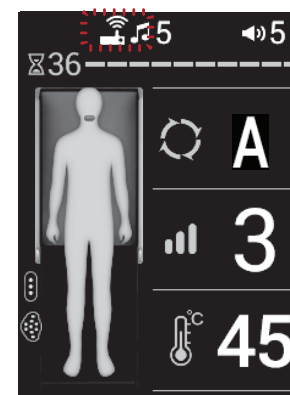


2. When the Internet connection is ready, the icon () will appear.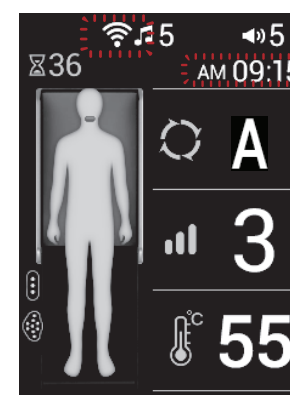

3. If the connection is made, the icon () will be displayed according to the intensity of the Internet signal, and the current time will also be displayed.
 - The current time will not appear if there is no Internet connection.


4. How to move the product

1 Disconnect the power cord, remote control, and external projector from the main body. Then, lift the head cushion and close the supporting mat. (To open the supporting mat, pull forward and then lower it.)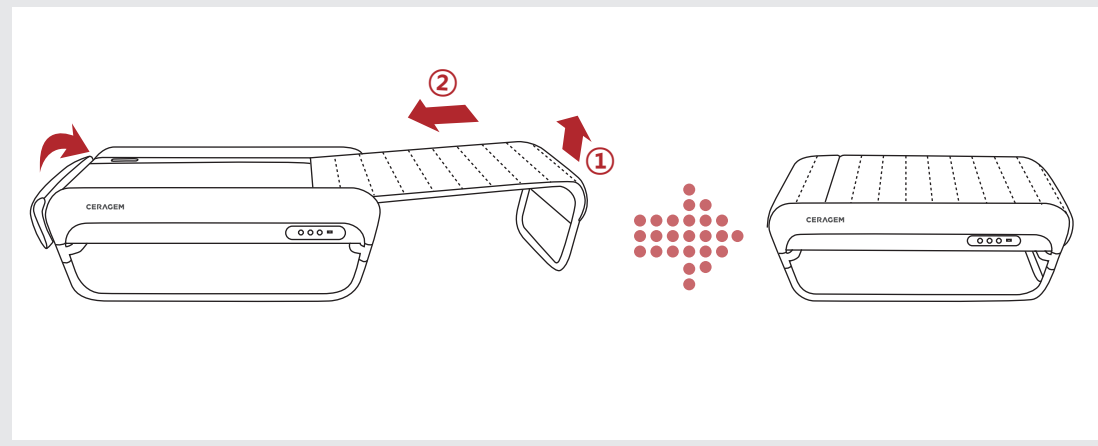

Caution When moving the product, please make sure that the remote control is turned off first before turning off the rest of the product. (After turning off the remote control, please wait 10 seconds before closing the supporting frame.)

2 Two people must hold both sides of the main body frame and lift and move horizontally.

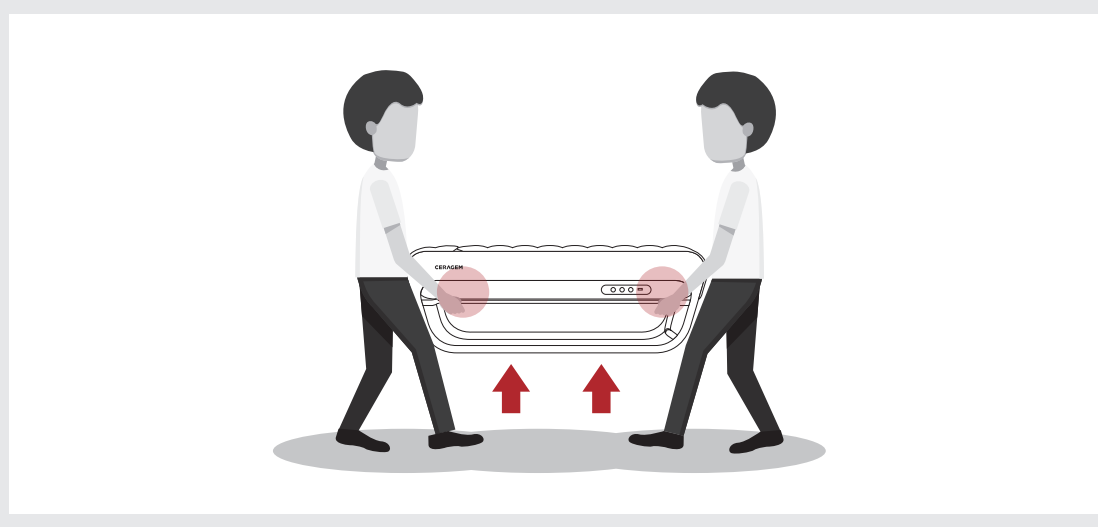Caution

- When lifting the product, please lift slowly while straightening your back and bending your knees.
- Be sure to prevent the supporting mat from opening while moving the product.

IV. Management and matters that require attention

1. Self inspection
2. Management techniques and storage methods
3. Check before A/S request
4. Diagnosis code message
5. Product specifications
6. Product warranty

1. Self inspection

Inspection points before use

1. Is the power cord connected to the socket?
2. Did you check whether the main body's power button is "ON"?
3. Is the remote control power and the LCD screen "ON"?
4. Did you check whether the projector's temperature has been set to the appropriate level?
5. Is there anything strange on the exterior?
6. Did you check whether there is no damage to the external projector cover?

Inspection points after use

1. Have the remote control and LCD screen been turned "OFF"?
2. Did you check whether main body's power button is "OFF"?
3. Is the power cord disconnected from the socket?

2. Management techniques and storage methods

- 1 Exterior of the product is a petroleum-based product and should not be exposed to organic solvents such as acetone, ballpoint pen, gasoline, thinner, sweat, hair mousse or hair spray.
- 2 Where there is direct contact with the head or skin, wipe with a dry cloth.
- 3 If there is contact with cosmetic products or volatile emulsions and is left for a long time, the surface may peel off so wipe with a dry cloth as soon as possible.
- 4 When washing the outer cloth, use a neutral detergent and wash alone by hand.
- 5 If there is contact with hot objects or heat, the leather will curl up so please use caution.
- 6 Avoid wearing clothes that may cause stains when using the product. (Please be careful as there is a risk of contamination caused by external friction.)
- 7 Store the product at room temperature, away from direct sunlight or humidity. (Discoloration and deterioration of the product may occur.)

3. Check before A/S request

Does the remote control turn on?

Q: Is the power switch (located at the side of the product) turned on?

A: Turn on the power switch.

Q: Is the power cord connected properly?

A: Connect the power cord again.

Q: Is the remote control connected properly?

A: Connect the remote control again.

The 3-sphere projector and the abdominal vibration projector do not operate.

Q: Are the 3-sphere projector and the abdominal vibration projector connected properly?

A: Connect the 3-sphere projector and the abdominal vibration projector properly.

Is the product malfunctioning?

Q: Can you move the internal projectors using the Up/Down key?

A: Internal projectors can be moved using the Up/Down key only in semi-automatic mode and manual mode.

Q: Can some of the remote control keys be operated?

A: Check the condition of the supporting mat.

Some of the remote control keys cannot be operated if the supporting mat is not completely opened.

Are error messages displayed on the remote control?

Q: Are the diagnosis code messages displayed on the remote control?

A: After checking the error messages, contact the Customer Support Center.

Never dismantle the product arbitrarily.

Why is the USB not recognized?

Q : Is a sound source in a USB device not playing back?

A : It takes time for the product to recognize a USB device. Check if it is connected correctly, wait, and try again.

4. Diagnosis code message

Wait!

Make sure to note down the 4-digit code and to contact the customer center in case the following message appears on the remote control while using the product.

Notify Ceragem and the competent organization (agency) in case of injury, accident (fire, personal injury, etc.) while using the product.

When a diagnosis code message is displayed, check the message and contact the Customer Service Center.

5. Product specifications

Model	CGM EB-1901
Article	Automatic Thermal Massager
Rated Voltage	220-240V~, 50/60 Hz
Power Consumption	190W
Environment temperature	10°C~40°C
Recommended height for body sensing	120 cm–186 cm
Playable files	MP3, WAV
USB format type	FAT, FAT32
Measurements	Fully opened: 700mmX2,040mmX450mm(±10mm) Folded: 700mmX1,280mmX450mm(±10mm)
Maximum Permissible Weight	135kg
Weight (Including box)	62kg(±3kg)
Manufacturer	CERAGEM Co., Ltd.

- * This appliance should not be used purpose of medical treatment
- * Please be advised that since this product was manufactured for domestic use, no after-service will be available when used abroad.
- * Due to constant quality improvement, design and specifications of the product are subject to change without notice.
- * Enter 'ceragem' at the Application Store and download the application.
- * Connection to wireless communication may be limited due to the distance between devices (speed), obstacles or other wireless devices.
- * Use of the application may be limited according to the model of the mobile phone or software version. (It is designed for smartphones. It may not operate smoothly in tablet PCs.)

6. Product warranty

Warranty

* For warranty purposes, Ceragem requires the following information:

Article	Automatic Thermal Massager	Model	CGM EB-1901
Date of Purchase		Serial No.	
Place of Purchase (Store)			

Free Services

If the product is used for business purposes, the warranty period is reduced by half.
Free service is provided for any normal-usage performance or functional failure occurring within 2 year of purchase. Product failure due to intentional negligence by the consumer (i.e. product failure due to repair of the product by an unauthorized technician; repair and/or alteration of the product by the consumer; and product damage or failure due to misuse of the product), will void the warranty.

Paid Services

In the absence of product failure:

- Incorrect installation of the product due to moving or relocation of the product;
- Occurrences that are not due to device defects or flaws
 - ▶ Slight noise, vibration, smell, appearance, feeling during use, etc.

Product failure due to negligence by the consumer:

- Product failure due to installation or incorrect use of the product by the consumer;
- Product damage or failure due to misuse of the product;
- Product failure due to use of an incorrect electric power supply;
- Product failure or damage due to external shock or dropping of the product;
- Product failure due to the use of consumables or products not specified by Ceragem;
- Product failure due to product repair performed by a person other than a Ceragem service agency technician;
- Product failure due to non-observation of the 'Matters that require attention' in the instruction manual.

Other instances:

- Product failure due to unforeseen disaster (flood, fire, earthquake, lightning, war etc.);
- Simple replacement of a consumable part that occurs naturally due to product use
 - ▶ Outer cloth, external projector cover, cables (external projector cable, remote control cable, connection cable between a sub-part and the main body, power cord), leather, etc.
- * Any service requested in cases where the product has not failed will be charged for. Please read the instruction manual carefully.
- * Keep this warranty, together with the instruction manual, in a safe place, since it is provided only in the country where the product is purchased and cannot be issued again.

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 **Conformite Europeenne Mark**

 Disposal of Old Electrical & Electronic Equipment
Applicable in the European Union and other European countries with separate collection systems.

This marking shown on the product or its literature, indicates that it should not be disposed with other household wastes at the end of its working life. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources. Household users should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can take this item for environmentally safe recycling.

V~ : AC (alternating current) voltage source
 I : On (power: connectino to the mains) O : Off (power: disconnection from the mains)

 : This way up  : Keep dry
 : Use no hooks  : Fragile (handle with care)
 : CLASS II EQUIPMENT

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